


Ypsoject® Silberhorn

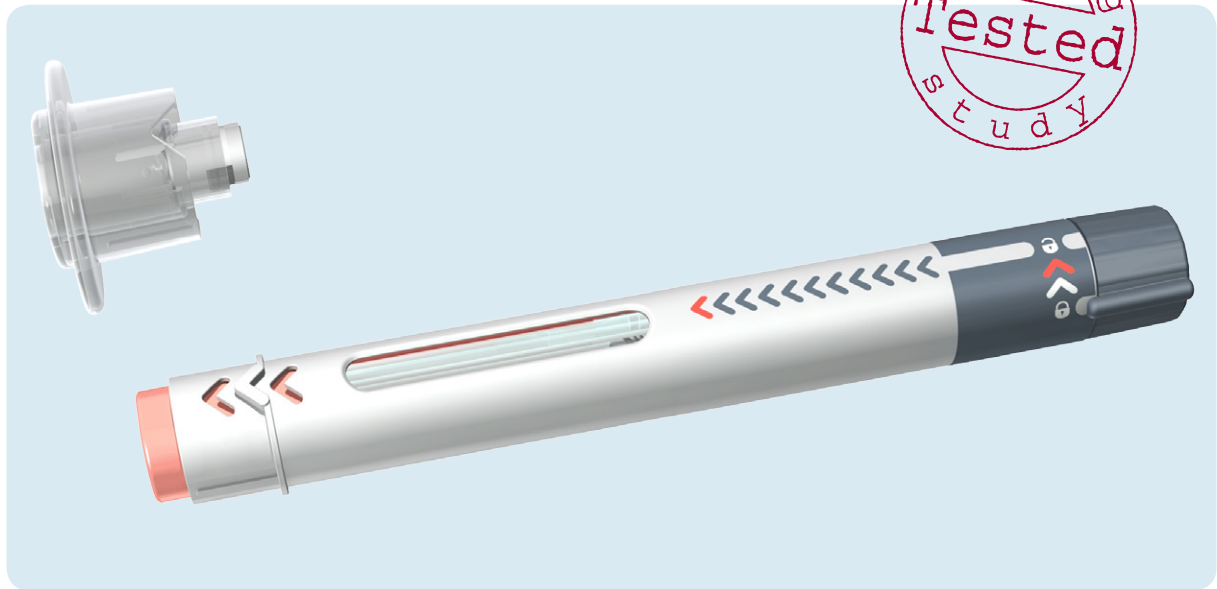

The Patient Controlled Easy to Use Disposable Auto-Injector


The easy to use, fully automated disposable auto-injector for all patient groups. The patented "controlled" mechanism is triggered using a simple "push-on-the-skin" injection technique. In comparative handling studies RA patients clearly preferred the Ypsoject® Silberhorn device from Ypsomed.

Multidose	Variable dose
Monodose	Fixed dose

Features and benefits

- Fully automatic needle insertion, injection and syringe retraction
- Easy to use, also for patients with impaired dexterity
- Secure "3-step" mechanism to prevent unintended triggering of the device
- Simple "push-on-the-skin" triggering mechanism
- Audible and tactile start and end-of-injection "CLICKS"
- Large viewing window to check the syringe and injection status
- Patient-controlled needle retraction after injection
- Full needle safety to minimise the risk of needle-stick injury
- Needle hidden from the patient's view at all times
- Optimum combination of patient-friendly features and device complexity

Main handling steps

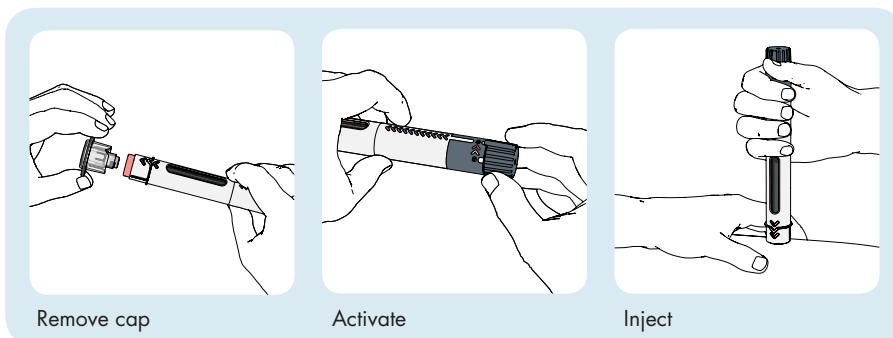


Remove cap


Activate

Inject

Auto-injector dimensions

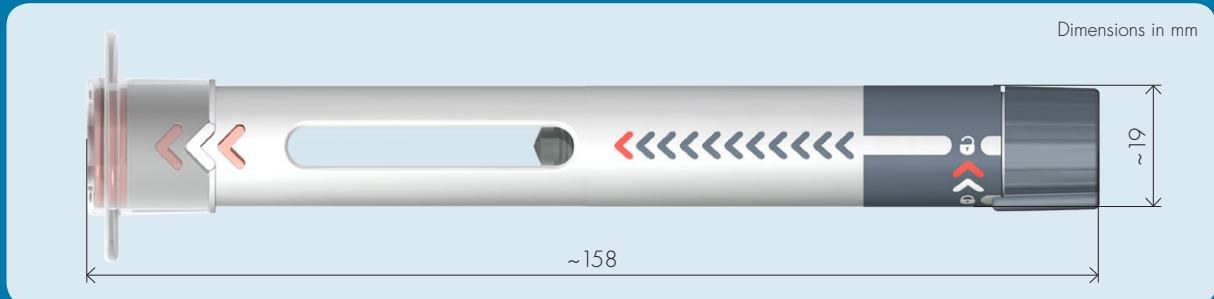

Syringe type

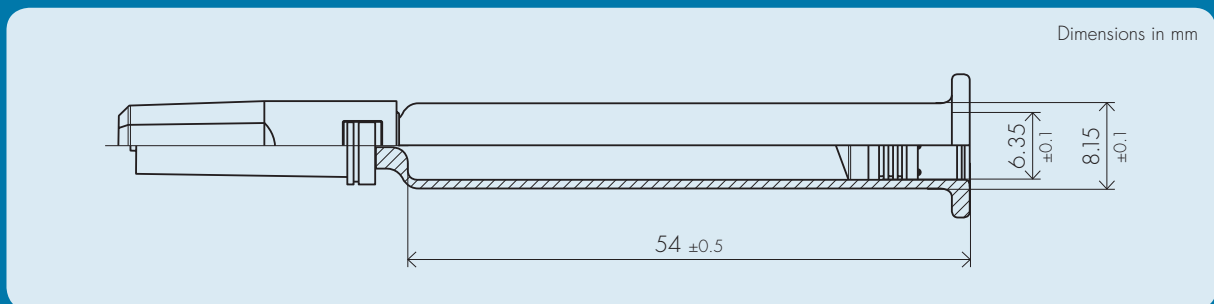

Standard 1ml long pre-fillable syringe with staked needles

COMPATIBILITY BETWEEN SYRINGE AND THE YpsoJect® Silberhorn AUTO-INJECTOR
REQUIRES VERIFICATION

Auto-injector specifications

- Deliverable volume of 1.0ml
- Compatible with rigid and rubber needle shields
- For subcutaneous injections up to 7mm insertion depth
- Viscosity range: 1–25cp depending on needle diameter and injection time